


INSTRUCTIONS FOR NON-CONTACT CORPSE WRAPPING WITH A BODY BAG

PART 1 :BAGGING

Matters needing attention :

1. Please follow the instructions.
2. Disposable product. Reuse is PROHIBITED.
3. Used ONLY for recovering corpses with NO VITAL SIGN.
4. Keep away from sharp objects.
5. Keep away from flames, heat, and direct sun.


1 Have the adhesive line face up

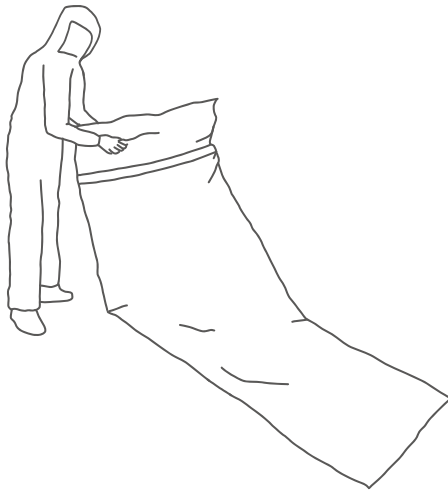


2 Flip the opening of the body bag outward to align the opposite closed end.



3 Extend your hand from the original opening into the lower space of the bag

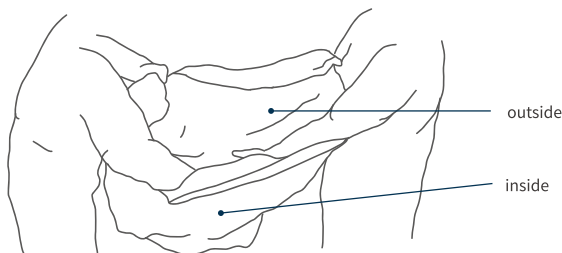


4 Put the corpse into the bag starting from the head of the corpse

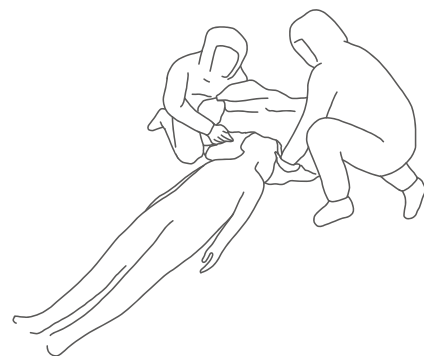

5 Continue and pull the opening down to the bottom




6 Wrap the corpse completely

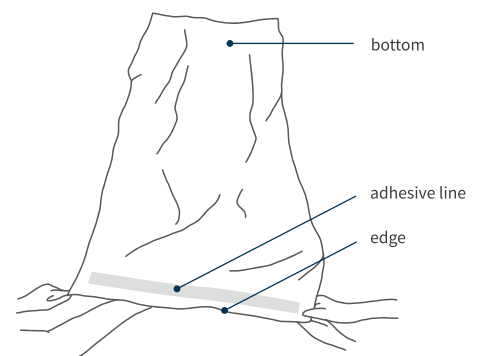


PART2: SEALING


7 Fold the edge of the opening in a zigzag shape to the top edge of the adhesive line

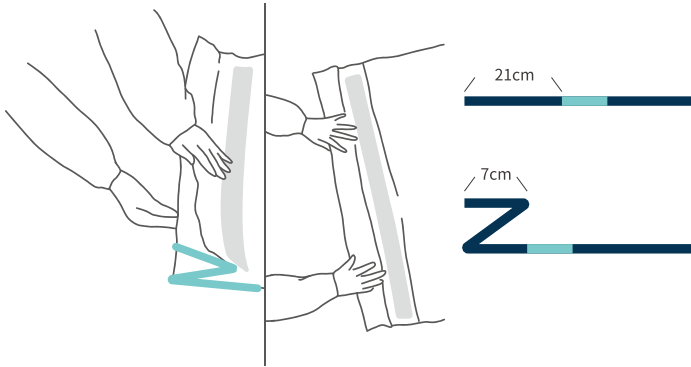


8 Fold in half to be 3.5 cm in width

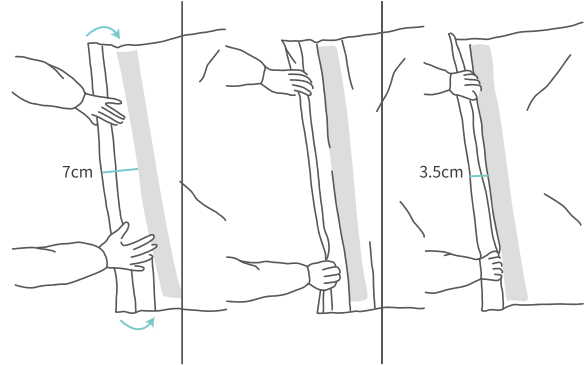


9 Tear off 2nd adhesive line

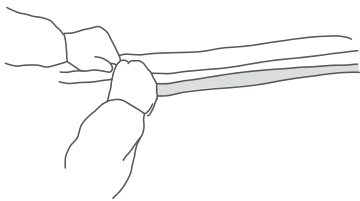


10 Fold-down along the top of the tape and press onto the adhesive line

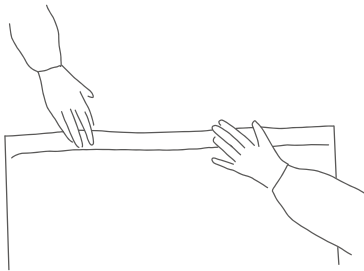


11 Grab the top left corner and fold it down to the right about 45 degrees

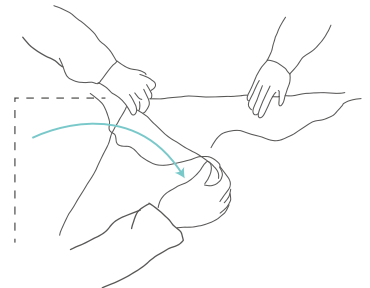

12 Grab the top right corner and fold it down to the left about 45 degrees

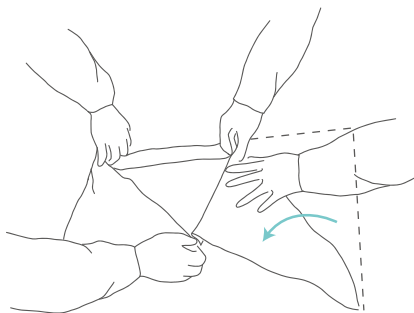

13 Knot the rope (flat or double bows are recommended)

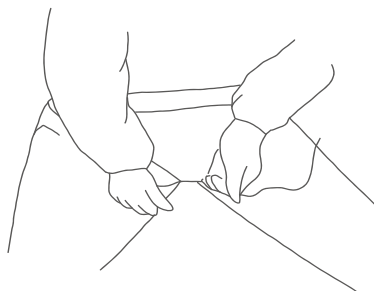

14 The seal is completed



15 Notice: DO NOT take the rope when lifting the side

